

ALL Z[®]

Solid Zirconia Screw Retained Hybrid Bridge



Doctor

Quick Reference



Laboratory

FIRST Appointment	Take Impression	Model Work . Fabricate Bite Block (3 working days)
SECOND Appointment	Take Bite Registration, select shade,/mold, impression of opposing dentition and current denture for study casts, mark midline, trim bite rim incisal edge, indicate high lip line	Articulate casts, set teeth in wax, fabricate implant verify jig and custom tray. (3 working days)
THIRD Appointment	Try-in wax setup, and verification jig, take final impression using included custom tray	Fabricate master cast, and optional Provisional Temp. (extra fee applies for temp) (8 working days)
Reset Appointment	Try in reset if necessary	Reset if necessary
FOURTH Appointment	Delivery of optional provisoinals	Fabrication of Final ALL Z 100% Zirconia Full-arch Implant Prosthesis (8 working days)
FIFTH Appointment	ALL Z 100% Zirconia Full-Arch Implant Prosthesis Final Delivery	